


**SAMPLE COLLECTION FOR FTA CARDS FOR INFECTIOUS BURSAL  
DISEASE VIRUS**

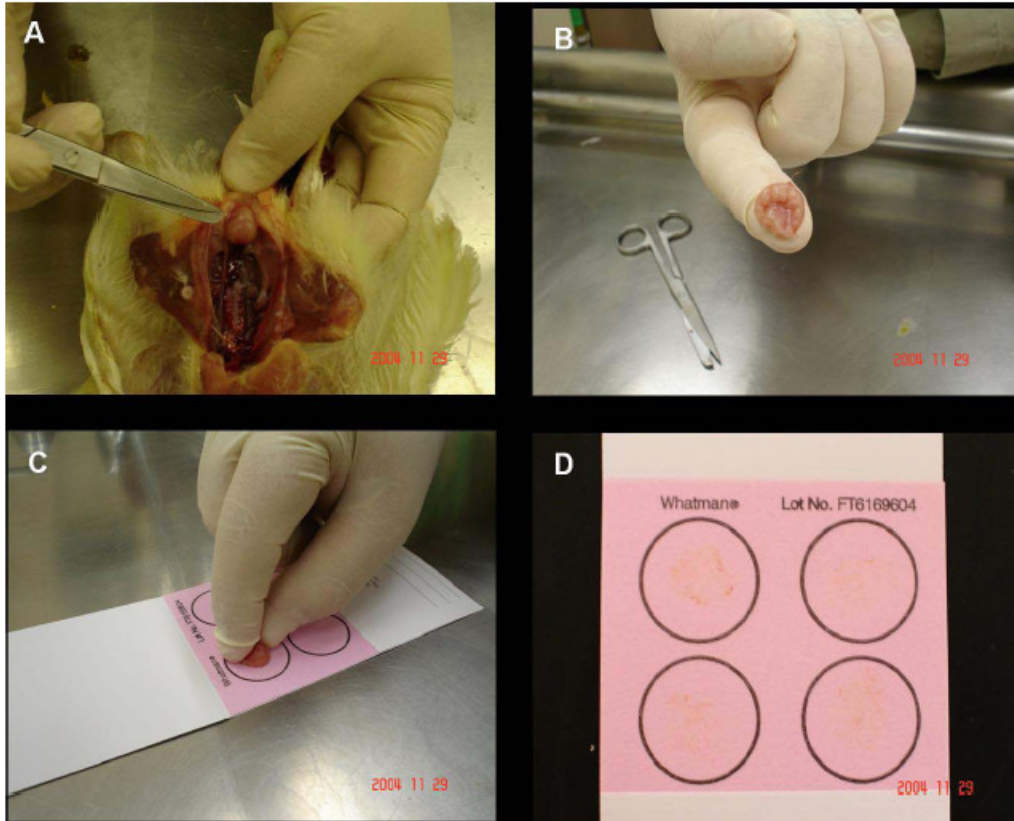

Bursas will be aseptically removed (A), cut in half (B) and transferred into the active circles of the FTA® cards (C and D). Samples transferred to the FTA® cards will be allowed to air-dry for 40 minutes at room temperature and will be ready to be sent by normal mail procedures. Please do not write or touch the pink area. Identify samples on the white space below the sample area.

Just place dried cards in a envelope and ship to:  
University of Georgia  
Poultry Diagnostic and Research Center  
Attn: Pedro Villegas  
953 College Station Rd.  
Athens, GA 30602  
USA

**EL LABORATORIO DE LA REGION**

Marcial Acharán N° 587- Urb. Las Quintanas Telef.: 44 208302 Telefax 44 249115 Celular 949930323

Trujillo-Perú

**Web: [www.microclin.com](http://www.microclin.com)**

**e-mail: [microclin@microclin.com](mailto:microclin@microclin.com)**